


Mist Tents/Mistogen/Model CAM 4

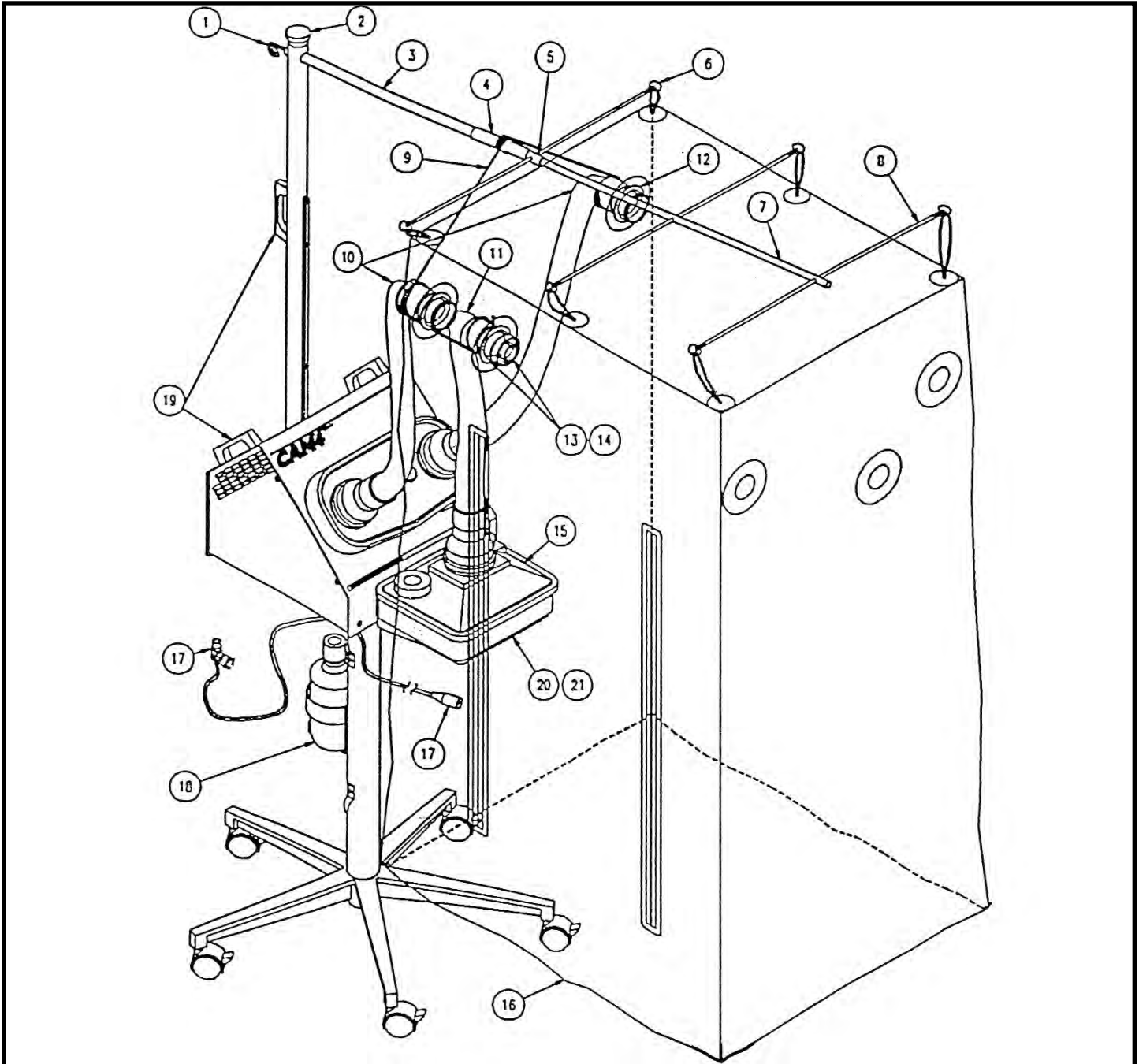

- | | | | |
|----------------|----------------------------|------------------|---------------------------|
| 1. #AW-T0793 | Pigtail Hook | 12. #AW-T15122 | Tent Ring |
| 2. #AW-T6321 | Knob with shaft | 13. #AW-T2438-1 | Tent Adapter, HV-12 |
| 3. #AW-T15146 | Base Bar | 14. #AW-T2436-1 | Tent Ring |
| 4. #AW-T0796 | Locking Sleeve | 15. #AW-T15150 | HV-12 Nebulizer for CAM4 |
| 5. #AW-T6353 | Socket Bar | 16. #AW-T6308 | Canopy |
| 6. #AW-T6397 | Canopy Tent Spring Hook | 17. #AW-T15144 | Grounding Cable Ass'y |
| 7. #AW-T6328 | Canopy Bar | 18. #AW-T6244 | Drain Bottle & Tube Ass'y |
| 8. #AW-T6330 | Tent Rod | 19. #AW-T2099.01 | Handle |
| 9. #AW-T15147 | Flex Hose Coil Support | 20. #AW-T15128 | Stability Strap (Hook) |
| 10. #AW-T15161 | Cooling Unit Flex Hose | 21. #AW-T15129 | Stability Strap (Loop) |
| 11. #AW-T15159 | Nebulizer 1-1/4" Flex Hose | | |
| #AW-T15160 | Nebulizer 2" Flex Hose | | |

Mist Tents/Mistogen/CAM 4 Con't.

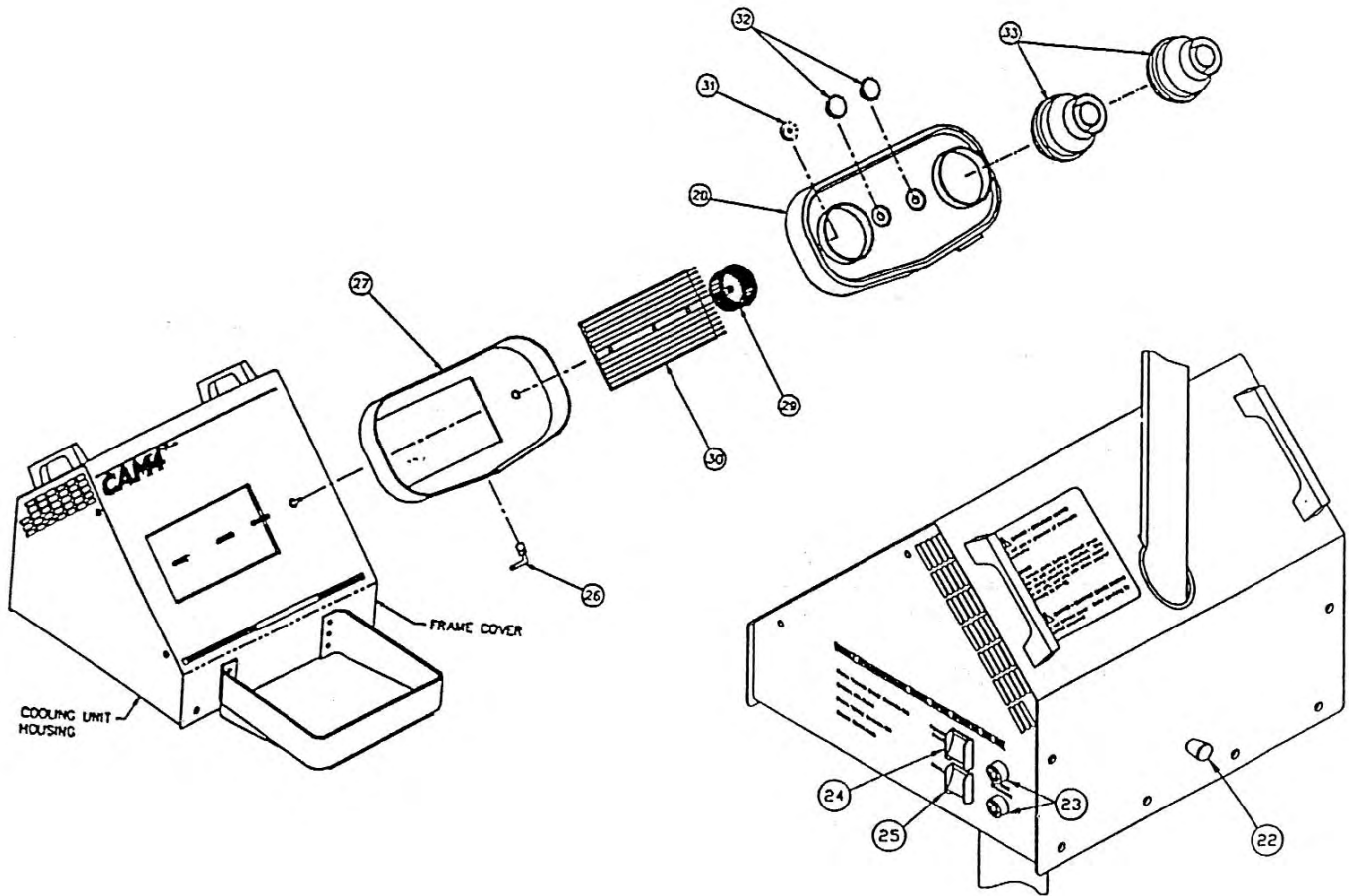


Figure #2

Figure #3

Item	Part No.	Description	Accessories & Miscellaneous
		Ref. Figure 3	
22	#AW-T15703	Plunger Pin	#AW-T6310 120VAC Circulation Motor
23	#AW-T12705	Circuit Breaker	#AW-T15710 230VAC Circulation Motor
24	#AW-T15704	Green DPST Switch, 110V	#AW-T6406 1000ml Reservoir w/Tube
	#AW-T15705	Green DPST Switch, 220V	#AW-T5870 Water feeder for HV-12
25	#AW-T15706	White DPST Switch, 110V	#AW-T14614 PCS-414 Medical air Compressor 120 V, 60 Hz
	#AW-T15707	White DPST Switch, 220V	#AW-T14618 PCS-414 Medical air Compressor 230 V, 50 Hz
		Ref. Figure 2	
26	#AW-T6141	Elbow Fitting	#AW-T15134 PCS-414 Deck
27	#AW-T15117	Air Exchanger Rear Cover	#AW-T15634 Accessories Stub Bar
28	#AW-T15118	Air Exchanger Front Cover	#AW-T1627.06 Terminal Strip
29	#AW-T6325	Circulating Impeller Fan	#AW-T15711 Cooling Fan (120 VAC)
30	Not Available	CoolXChanger	#AW-T15712 Cooling Fan (230 VAC)
31	#AW-T0755	Knurled Knob	
32	#AW-T15708	Fluted Knob	
33	#AW-T13242-1	3" x 2" Adapter	
	#AW-T2437-1	Reducer	
	#AW-T1346	Nylon Screw, #6-32	


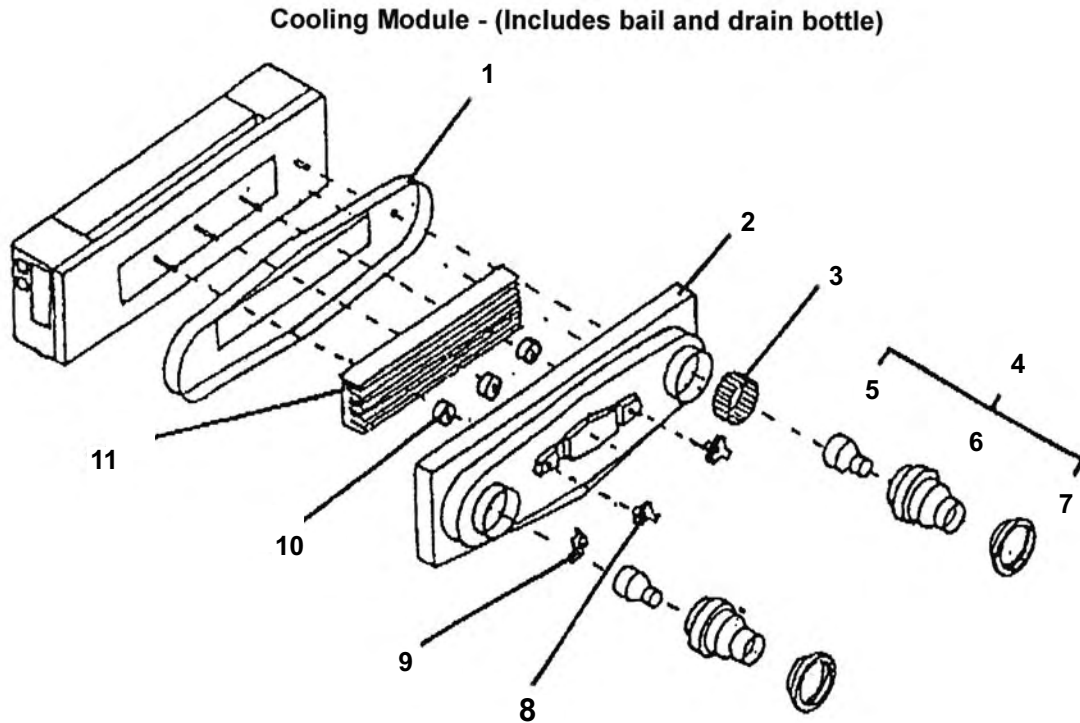


Figure #3

<u>Item</u>	<u>Product Number</u>	<u>Description</u>
XXX	#AW-6375	Cool Module 230V 50/60Hz 1.5A
XXX	#AW-6335	Cool Module 115V 50/60Hz 3A
1.	#AW-6303	Rear Cover
2.	#AW-6302	Cover
3.	#AW-6325	Impeller
5.	#AW-2437	Reducer
6.	#AW-2443	Adapter, 2" x 3"
7.	#AW-2436	Clamp/Tent Ring
8.	#AW-6307	Cover Knob
9.	#AW-6141	Drainage Elbow
10.	#AW-0755	Metal Knob
11.	#AW-6342	Cool XChanger

Mist Tents/Timeter/CAM[®] - 3

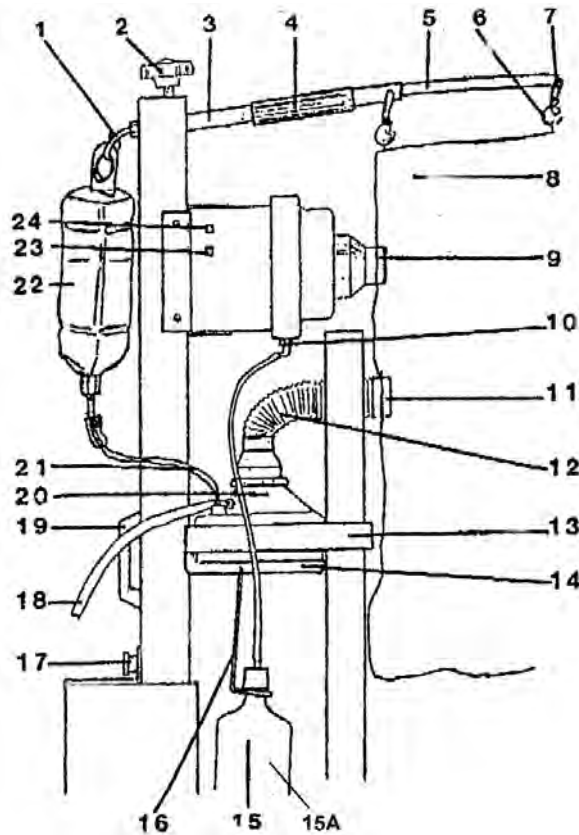

Parts referenced in Figure # 2

ITEM	PART #	DESCRIPTION	ITEM	PART #	DESCRIPTION
			15b.	#AW-6221	Tube Connector for #AW-8504
			15c.	#AW-8504	Cap for #AW-8505 Drain Bottle
			16.	#AW-6336	Hanger (Bail) for Drain Bottle
			17.	N/A	Locking Pin
1.	#AW-0793	Reservoir Hook	18.	#AW-2235.12	High Pressure Hose
2.	#AW-6321	Knob w/shaft	19.	#AW-2099	Handle
3.	#AW-6749	Stub Bar Assembly	20.	#AW-6380	HV12 Nebulizer for CAM [®] 3
4.	#AW-0796	Sleeve for Stub Bar	20a.	#AW-6692	Right Angle Adapter For #AW-6380
5.	#AW-6328	Canopy Bar	21.	#AW-6409	Tube Set for #AW-6406 Reservoir
6.	#AW-6397	Tent Spring Hook	22.	#AW-6406	1000ml Plastic Reservoir w/hanger
7.	#AW-6330	Tent Rod	23a.	#AW-6366	White Switch
8.	#AW-6308	Canopy	23b.	#AW-6377	Lamp, White Circulation Switch
9.	--	See Fig. 3 parts.	23c.	#AW-11480	White Circulation Switch Cover
10.	--	See Fig. 1 parts.	24a.	#AW-6365	Green Switch
11.	#AW-2438	Tent Adapter	24b.	#AW-6378	Lamp, Green Switch & Indicator
12.	#AW-6765	Tube Assembly, Coaxial	24c.	#AW-11479	Green Cooling Switch Cover
13.	#AW-6341	Stability Strap			
14.	#AW-6668	L-Bracket			
14a.	#AW-2641	Hoop, HV-12 Nebulizer			
15.	#AW-6244	Drain Bottle/Tube Assembly (Incl: Bottle #AW-8505, Tubing #AW-6263, Cap #AW-8504, Tube Connector #AW-6221			
15a.	#AW-8505	1000ml Drain Bottle w/o hardware			