


Nebulizers/Mistogen


MODEL MP - 500 COMPU-NEB ELECTRONIC NEBULIZER

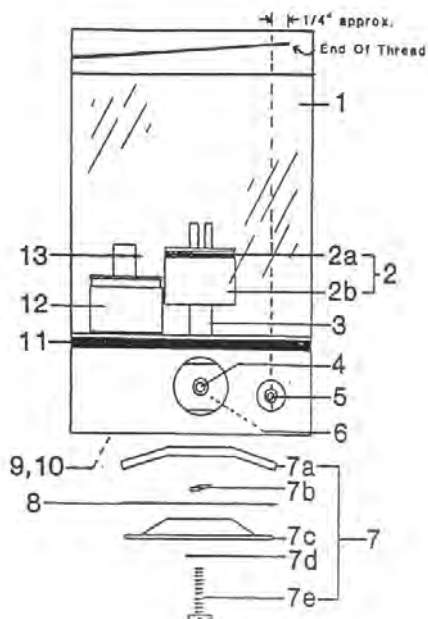

TRANSDUCER PARTS

Item	Part Number	Description
1.	#AW-3768	Reservoir
2.	#AW-3850	Float Valve Assembly
2a.	#AW-1681	Retaining Ring Only
2b.	#AW-3862	Float Only
3.	#AW-3858	Pipe, Water
4.	#AW-4147	Water Connector
5.	#AW-4524	RF Connector w /internal wire
	#AW-4444	RF Connector Only
6.	#AW-0544	O-ring Only, 014
7.	#AW-4172	Inspection Cover Assembly
7a.	#AW-4171	Clamp, Inspection Cover
7b.	#AW-1005	Lock Washer, #8
7c.	#AW-4170	Inspection Cover
7d.	#AW-0696	Washer, #8 Fiber
7e.	#AW-0974	Screw, Inspection Cover 6-32 x .625
8.	#AW-4161	Gasket, Inspection Cover
9.	#AW-11353	Crystal Kit
10.	#AW-6030	Reed Switch Kit
11.	#AW-3958	Gasket, 2.68 x 3.03
12.	#AW-3609	Switch Float Assembly
13.	#AW-3610	E-Ring

CHASSIS PARTS

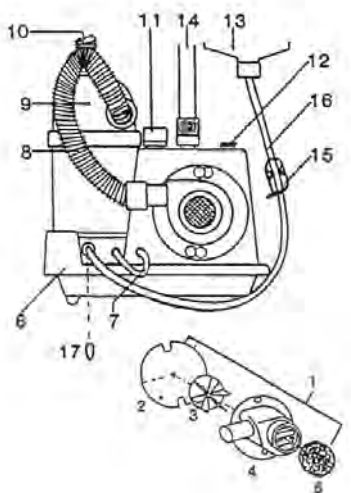

Item	Part Number	Description
1.	#AW-5515	Blower Housing Assembly Items 2,3,4,5
2.	#AW-4112	Shield
3.	#AW-5518	Impeller
4.	#AW-5519	Housing
5.	#AW-4129	Filter
6.	#AW-11493*	Cover Assembly
7.	#AW-4445.01	RF Cable
8.	#AW-3501	Medical air Tube
9.	#AW-3824	Transducer Cover
10.	#AW-3081	Aerosol Tubing
11.	#AW-4947	Handle
12.	#AW-2589	Cover Nut
13.	N/A	Reservoir
14.	#AW-2550	IV Post, Telescoping
15.	#AW-4149	Water Shut-off Clamp
16.	#AW-N/A	Fluid Tube Set
17.	#AW-5293	Water Cap


Part Number	Description
#AW-513-001	Fuse
#AW-4578	Capacitor, 5100 μ F, 40v
#AW-11450*	Potentiometer, 10K
#AW-10512	Power Switch
#AW-10513	RF Module
#AW-11451*	Boot, Power Switch
#AW-11453	Boot, Mist Control Knob
#AW-11483*	Lamp, solid state T-1-3/4
#AW-11494*	Mist Control Knob

Older Version Parts List

Part Number	Description
#AW-1456*	Lamp, T -1 3/4
#AW-8318*	Boot, Switch Power
#AW-10511*	Potentiometer, 10K (Mist Control)
#AW-4456*	Mist Control Knob
#AW-2613*	Cover (w/Labels)

***Check the Serial Number Plate on your COMPU-NEB™ before ordering this part. If the number is "pre-" M16A, order the part from the "OLDER VERSION PARTS LIST."**

MODEL HV - 12 - HIGH VOLUME PNEUMATIC NEBULIZER

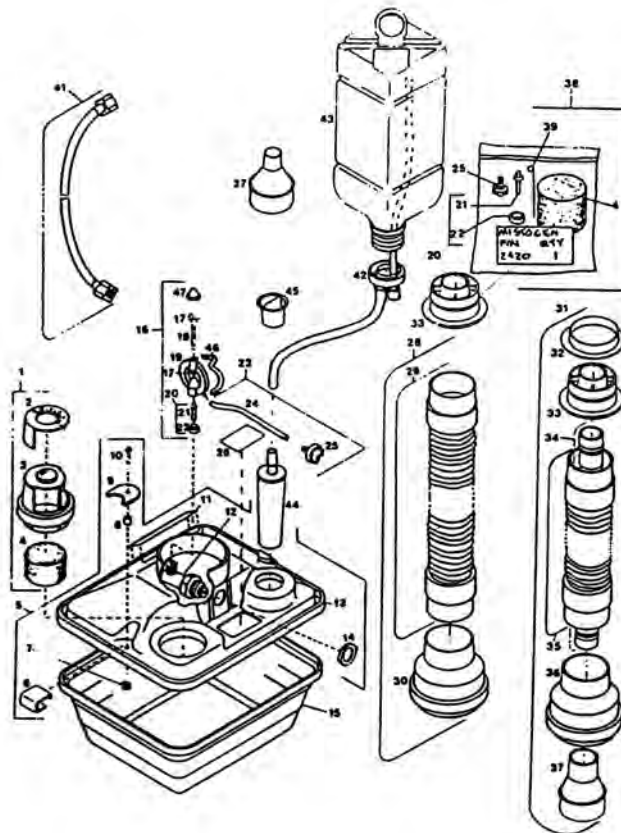

REPLACEMENT PARTS

Item	Part Number	Description
1.	#AW-2461	Fill Plug Assembly (Incl. 2-4)
2.	#AW-2447	Dilution Control Cap
3.	#AW-2446	Fill Plug Body
4.	#AW-2462	Filter
5.	#AW-5049	Cover Replacement
6.	#AW-2412.01	Latch (2)
7.	#AW-1696.832	Lock Nut, 8-32 (2)
8.	#AW-1695.31	Spacer, #8 x .250 (2)
9.	#AW-2991	Clip Retainer (2)
10.	#AW-1424.75	Machine Screw 8-32 (2)
11.	#AW-2448	Signature Decal
12.	#AW-2477	Supply Elbow
13.	#AW-2425.01	Cover w/o hardware
14.	#AW-1396	Jam Nut
15.	#AW-2410	Basin
16.	#AW-2404	Jet Assembly (Incl. 17-22, 46-47)
17.	#AW-S101-101-010D	Jet Jam Nut, 10 - 32 (2)
18.	#AW-2691	Baffle Rod
19.	#AW-2406	Jet Holder
20.	#AW-16520	Jet With O-ring (Incl. 21, 22)
22.	#AW-1568.08	O-Ring
23.	#AW-2418	Suction Tube Assy (Incl. 24-25)
24.	#AW-2419.60	Tube, .1 ID x 6"

REPLACEMENT PARTS

Item	Part Number	Description
25.	#AW-6179	Foot Filter, 149 MICRONS
26.	#AW-2449	Data Decal
27.*	#AW-2432	2" x 3/4" Reducer
28.*	#AW-2463	Aspirating Tube Set (Incl. 29, 30)
29.*	#AW-4098	2" x 36" Tube Assembly
30.*	#AW-2443	3" x 2" Adapter
31.*	#AW-12732	Coaxial Tube Set (Incl. 32-37)
32.*	#AW-2436	Tent Ring
33.*	#AW-2438	2" x 1 1/4" Tent Adapter
34.*	#AW-2444.018	1 1/4" x 18" Tube Assembly
35.*	#AW-2209.018	2" x 18" Tube Assembly
36.*	#AW-2443	3" x 2" Adapter
37.*	#AW-2437	2" x 1 1/4" Reducer
38.	#AW-2420	Service Kit (Incl. 4, 20,25,39)
39.	#AW-0019	Cleaning Wire
41.	#AW-2235.12	High Pressure Hose
42.**	#AW-5007	Fluid Tube Connector 12.5
43.**	#AW-6406	Bottle With Holder
44.	#AW-5870	Feeder
45.	#AW-6334	Pull Plug
46.	#AW-6631	Antistatic Harp
47.	#AW-1021	Rounded Nut
***	#AW-12733	14" Coaxial Tube Set For CAM®3
*	#AW-11523	HV-12 Grounding Kit

*Not included with all models

**Optional for all models

***Not Shown