**DESCRIPTION:**  
The Mercury Flow-Safe Continuous Positive Airway Pressure (CPAP) device is a respiratory aid intended for use with a facemask and gas supplying device to elevate pressure in the patient's lungs.

**CONNECTIONS:**  
- Standard oxygen tubing nipple  
- Patient connection ISO 5356-1 - 15mm taper female and 22mm taper male

**INDICATIONS FOR USE:**  
The Mercury Flow-Safe CPAP device is intended to provide CPAP to spontaneously breathing patients in the hospital and pre-hospital environment.

**WARNINGS:**  
- Do not allow smoking or use unit near sparking equipment, open flame, oil or other flammable chemicals.  
- Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

**CAUTIONS:**  
- Federal (USA) law restricts this device to sale by or on the order of a physician.  
- Do not clean, soak, rinse or sterilize.  
- Reuse of this device may pose a risk of cross-contamination and the device may not perform as intended.  
- In the event of undesirable flow rate from oxygen source, simply remove the device and place on supplemental oxygen per protocol.  
- Use of the Flow-Safe with non-back pressure compensated flow devices may affect input gas liter flow. Always verify delivered CPAP pressure on manometer.  
- This device is assembled with PVC components which include low levels of a phthalate as indicated. Consideration should be used when treating children or pregnant or nursing women.


**CONTRAINDICATIONS:**  
- Respiratory Arrest  
- Unconscious  
- Cardiogenic Shock  
- Pneumothorax  
- Facial Anomalies  
- Facial Trauma  
- Airway Obstruction

**DIRECTIONS FOR USE:**  
1. Connect O<sub>2</sub> tubing nipple to gas source.  
2. Secure the face mask snugly to patient's face using head harness.  
3. Slowly increase gas flow to 6 or 8 LPM. Check mask fit to patient and device connections for leaks.  
4. Adjust the flowmeter until desired pressure is obtained. Flow of 12 - 14 LPM is required to reach CPAP pressure of 8.5 - 10 cm H<sub>2</sub>O.  
5. Do not exceed 30 LPM.  
6. Patient SaO<sub>2</sub> should be monitored using a pulse oximeter.  
7. Dispose of according to facility protocol.

**MEASURING PRESSURE:**  
- Pressure relief limits maximum CPAP pressure to 25 cm H<sub>2</sub>O @ 25 LPM.  
- Do not exceed pressure limit of manometer (25 cm H<sub>2</sub>O).  
- Manometer accuracy ± 3 cm H<sub>2</sub>O up to 15 cm H<sub>2</sub>O and ± 5 cm H<sub>2</sub>O over 15 cm H<sub>2</sub>O.

**OPERATING ENVIRONMENTAL LIMITS:** -18° C to +50° C  
**STORAGE ENVIRONMENTAL LIMITS:** -40° C to +60° C

**MR CONDITIONAL:**  
Static magnetic field of 3.0 Tesla or less  
Spatial Gradient of 3000 gauss/cm or less

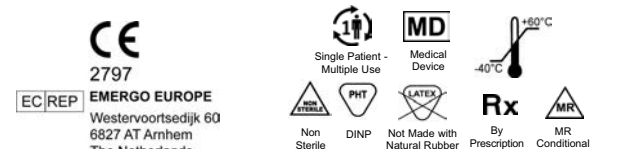© Registered U.S. Trademark of Mercury Enterprises, Inc. Patent # US 8,522,618 B1; US 9,370,635 B2; US 11,383,054 B2; Other Patents Pending 1/2023 #83-900-0390 Rev. 7

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