Mercury Medical logo and address: 11300 - 49th Street North, Clearwater, Florida 33762-4807 USA

(en)

Flowsafe II EZ Disposable CPAP System

DESCRIPTION: The Mercury Flow-Safe II EZ Continuous Positive Airway Pressure (CPAP) device is a respiratory aid intended for use with a face mask, nebulizer and gas supplying device to elevate pressure in the patient's lungs...

CONNECTIONS: Standard oxygen tubing nipple, Patient connection ISO 5356-1 - 15mm taper female and 22mm taper male, Nebulizer connection 18mm taper female and 22mm taper male

INDICATIONS FOR USE: The Mercury Flow-Safe II EZ CPAP device is intended to provide CPAP to spontaneously breathing patients in the hospital and pre-hospital environment.

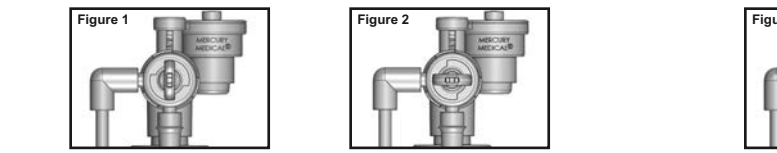
WARNINGS: Do not allow smoking or use unit near sparking equipment, open flame, oil or other flammable chemicals. Any serious incident that has occurred in relation to the device should be reported to the manufacturer...

CAUTIONS: Federal (USA) law restricts this device to sale by or on the order of a physician. Do not clean, soak, rinse or sterilize. Reuse of this device may pose a risk of cross-contamination...

CONTRAINDICATIONS: Respiratory Arrest, Unconscious, Cardiogenic Shock, Pneumothorax, Facial Anomalies, Facial Trauma, Airway Obstruction

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MEASURING PRESSURE: Pressure relief limits maximum CPAP pressure to 25 cm H2O @ 25 LPM. Do not exceed pressure limit of manometer (25 cm H2O). Manometer accuracy is 3 cm H2O up to 15 cm H2O and 5 cm H2O over 15 cm H2O.

MEASURING PRESSURE: Pressure relief limits maximum CPAP pressure to 25 cm H2O @ 25 LPM. Do not exceed pressure limit of manometer (25 cm H2O).

Table with 5 columns: FLOWMETER SETTING, CPAP PRESSURE, FLOW THROUGH EZFLOW MAX, OUTPUT, and RX (mg/hour) for various treatment durations.

EXAMPLE: To deliver 10 mg/hour of 5 mg/mL medication at 10 L/min; 1.5 Hour Duration: Place 3 mL of medication + 21 mL of Saline into nebulizer reservoir and run for 1.5 hours.

CAUTION: The listed output is nominal value only, actual output may vary depending on device, accuracy of flowmeter, and the existence of tubing connection leaks. Follow up output checks are recommended and flowmeter adjustments may be necessary.

CE, EC REP, EMERGO EUROPE logos and regulatory information for the English version.

Mercury Medical logo and address: 11300 - 49th Street North, Clearwater, Florida 33762-4807 USA

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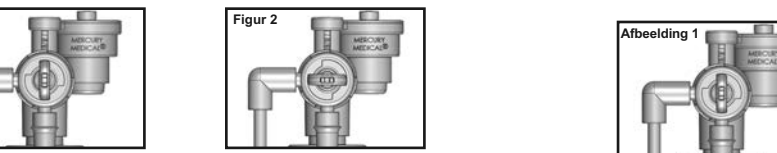
MEASURING PRESSURE: Pressure relief limits maximum CPAP pressure to 25 cm H2O @ 25 LPM. Do not exceed pressure limit of manometer (25 cm H2O). Manometer accuracy is 3 cm H2O up to 15 cm H2O and 5 cm H2O over 15 cm H2O.

MÅLING AF TRYK: Trykløst ventileret begrænser maksimalt CPAP-tryk til 25 cm H2O @ 25 LPM. Do not exceed pressure limit of manometer (25 cm H2O).

Table with 5 columns: INDSTILLING AF FLOWMETER, CPAP-TRYK, FLOW GENNEM EZFLOW MAX, OUTPUT, and RX (mg/time) for various treatment durations in Danish.

EXAMPLE: For at levere 10 mg/ur af 5 mg/ml medicin ved 10 l/min; 1,5 timer varighed: Anbring 3 ml medicin + 21 ml saltvandsopløsning i forstøvers beholder, og kør i 1,5 timer.

CAUTION: The listed output is nominal value only, actual output may vary depending on device, accuracy of flowmeter, and the existence of tubing connection leaks. Follow up output checks are recommended and flowmeter adjustments may be necessary.

CE, EC REP, EMERGO EUROPE logos and regulatory information for the Danish version.

Mercury Medical logo and address: 11300 - 49th Street North, Clearwater, Florida 33762-4807 USA

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MEASURING PRESSURE: Pressure relief limits maximum CPAP pressure to 25 cm H2O @ 25 LPM. Do not exceed pressure limit of manometer (25 cm H2O). Manometer accuracy is 3 cm H2O up to 15 cm H2O and 5 cm H2O over 15 cm H2O.

MEASURING PRESSURE: Pressure relief limits maximum CPAP pressure to 25 cm H2O @ 25 LPM. Do not exceed pressure limit of manometer (25 cm H2O).

Table with 5 columns: INSTELLING STROOMMETER, CPAP-DRUK, STROOM DOOR EZFLOW MAX, OUTPUT, and RX (mg/uur) for various treatment durations in Dutch.

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CAUTION: The listed output is nominal value only, actual output may vary depending on device, accuracy of flowmeter, and the existence of tubing connection leaks. Follow up output checks are recommended and flowmeter adjustments may be necessary.

CE, EC REP, EMERGO EUROPE logos and regulatory information for the Dutch version.

Mercury Medical logo and address: 11300 - 49th Street North, Clearwater, Florida 33762-4807 USA

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Table with 5 columns: IMPOSTAZIONE FLUSSIMETRO, CPAP-TRYK, FLOW GENNEM EZFLOW MAX, OUTPUT, and RX (mg/ORA) for various treatment durations in Italian.

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CE, EC REP, EMERGO EUROPE logos and regulatory information for the Italian version.

Mercury Medical... 11300 - 49th Street North, Clearwater, Florida 33762-4807 USA, www.mercurymed.com

Flowsafe II EZ CPAP SYSTEM FOR ENGLANDS BRUK

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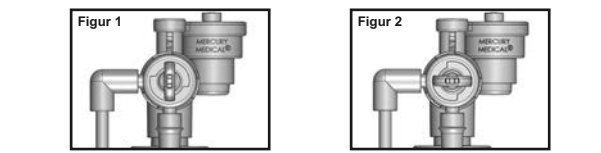
Table with columns for flow rate (14-15, 23-24) and pressure (4-5, 9-10) showing flowmeter readings for different flow rates.

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EMERGO EUROPE logo and various certification marks (CE, ISO, etc.) for the device.

Mercury Medical... 11300 - 49th Street North, Clearwater, Florida 33762-4807 USA, www.mercurymed.com

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