Mercury Medical
Superior Solutions™
11300 - 49th Street North
Clearwater, Florida 33762-4807 USA
www.mercurymed.com

**Flow-Safe II® & Flow-Safe II EZ®
Apnea Alarm Accessory**

(en)

Single-Patient-Use

CONTENTS:
In-Line Tee
Luer Lock Sampling Tubing

INSTRUCTIONS:
Insert In-Line Tee between mask and Flow-Safe II & Flow-Safe II EZ device. Securely connect Luer Lock Sampling Tubing to In-Line Tee and the anesthesia system's gas analyzer.
Dispose of according to facility protocol.

CAUTIONS:
1. Single use only. Do not sterilize or re-use to reduce the risk of possible transmission of infection.
2. During use, the patient's condition must be continuously monitored.
3. Federal (USA) law restricts this device to sale by or on the order of a physician.
4. This device is assembled with PVC components which include low levels of a phthalate as indicated. Consideration should be used when treating children or pregnant or nursing women.

WARNINGS:
- For apnea alarm triggering only. If used for capnometry or capnography, numeric value or wave form may be attenuated. Do not use as a diagnostic tool. For accurate capnometry or capnography, a nasal sampling cannula is recommended.
- Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

1/2023 #83-900-0482 Rev. 2

Mercury Medical
Superior Solutions™
11300 - 49th Street North
Clearwater, Florida 33762-4807 USA
www.mercurymed.com

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