INDICATIONS FOR USE:

An oropharyngeal airway (OPA) is a device intended to maintain a gas pathway through the oral cavity and pharynx.

DIRECTIONS FOR USE:


- Choose correct size of OPA. To size, use the device itself as a measure. Place it on the patient's cheek with the flange parallel to front teeth; the tip of the oropharyngeal airway should reach no further than the angle of the jaw. If the airway is too long, it could obstruct breathing by displacing the tongue against the oropharynx. If it's too short, it won't be able to hold the tongue away from the pharynx, and patency won't be restored.
- Position patient supine with neck fully extended.
- Open mouth and pull chin forward.
- Suction the patient's mouth and pharynx to remove any secretions.
- Insert into the patient's mouth upside down.
- Once contact is made with the back of the throat, the airway is rotated 180 degrees, allowing for easy insertion, and assuring that the tongue is secured. (An alternative method for insertion, the method that is recommended for OPA use in children and infants, involves holding the tongue forward with a tongue depressor and inserting the airway right side up.)
- The device is removed when the patient regains swallow reflex and can protect their own airway. Remove by pulling on it without rotation.

WARNINGS:

- If patient isn't breathing spontaneously or is breathing inadequately, initiate artificial ventilation using a mouth-to-mask or bag valve mask technique.
- Tissue irritation and ulceration of the mucous membranes of the mouth can occur if the airway is left in place for prolonged use.
- Do not use on patient who is actively seizing.

CAUTIONS:

- If the patient gags or appears to be gasping for air after insertion, remove immediately. Recheck the size before attempting reinsertion.
- Taping an oropharyngeal airway in place can lead to aspiration if the patient starts to gag or cough and can't expel the device.
- The Guedel Airway is a tubular device with its central lumen which can be used for suctioning but can't support an ET tube.
- Remove dentures (if applicable)
- The presence of stridor, gasping, or snoring may be an indication of displacement of the airway or be a sign of increasing respiratory distress.

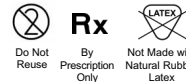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DIRECTIONS FOR USE:

Oropharyngeal airway

1. Position patient supine with neck fully extended
2. Open mouth and insert airway midline
3. Pull chin forward

Guide for flexible fiberoptic laryngoscopy or bronchoscopy

1. Administer appropriate anesthetics
2. Open mouth and insert airway midline
3. Pull chin forward
4. Pass lubricated laryngoscope or bronchoscope through airway past epiglottis, through larynx into trachea

Guide for fiberoptic assisted endotracheal tube insertion

1. Slide appropriately sized and lubricated endotracheal tube to top of lubricated fiberoptic scope
 - 7.5 mm ID endotracheal tube for 9 cm airway
 - 8.5 mm ID endotracheal tube for 10 cm airway
2. Insert airway (per oropharyngeal directions above)
3. Pass fiberoptic scope through airway and visualize distal airway tip
4. Advance fiberoptic scope until epiglottis is in view
5. Slide endotracheal tube down over fiberoptic scope into trachea (rotate proximal end of endotracheal tube 90 to 180 degrees if required)

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