

# StatCO<sub>2</sub>®, Mini StatCO<sub>2</sub>® & Neo-StatCO<sub>2</sub> <Kg®

Performs for 24 Hours in 100% Humidity!



No Pull Tab

## Neo-StatCO<sub>2</sub> <Kg®

For patients within  
0.25 kg - 6 kgs body weight



NEW Red Pull Tab for better visibility activates device

## Mini StatCO<sub>2</sub>®

For patients within  
1 kg - 15 kgs body weight

## StatCO<sub>2</sub>®

For patients over  
15 kgs body weight



### StatCO<sub>2</sub>/Mini StatCO<sub>2</sub>/Neo-StatCO<sub>2</sub> <Kg

Blue	No Carbon Dioxide is present
Green	Between 1.0% to 2.0% CO <sub>2</sub> is present
Yellow/Green	At intermediate CO <sub>2</sub> concentrations of 2.0% - 5.0%
Yellow	5.0% or more CO <sub>2</sub> is present
Permanent Yellow Color	A damaged indicator

### PRIOR TO USE:

- Check package expiration date. If expired DISCARD DEVICE.
- Remove StatCO<sub>2</sub>, Mini StatCO<sub>2</sub> and Neo-StatCO<sub>2</sub> <Kg from package.
- Compare color of indicator to Blue color label on detector.  
*(No Blue color label on the Neo-StatCO<sub>2</sub> <Kg)*
  - If colors match, continue to use StatCO<sub>2</sub>, Mini StatCO<sub>2</sub> and Neo-StatCO<sub>2</sub> <Kg.
  - If colors do not match, DISCARD DEVICE.
- Connect StatCO<sub>2</sub> and Mini StatCO<sub>2</sub> between ET tube and breathing device and pull paper strip straight out to activate CO<sub>2</sub> Detector  
*(Pull tab on StatCO<sub>2</sub>/Mini StatCO<sub>2</sub> only).*
- Connect Neo-StatCO<sub>2</sub> <Kg between ET tube and breathing device  
*(No pull tab).*
- Provide 6 breaths of normal tidal volume ventilation.
- After 6 breaths and on full expiration, compare indicator color to ranges of colors on detector face.

