# Tips on Using *The Elevation Pillow*

- 1) Place the Elevation Pillow (EP) at the head of the O.R. table and cover the EP with its **form fitted impermeable barrier cover**, (this cover provides added patient comfort and helps to keep the EP clean). **Place a standard intubation pillow or foam donut on the upper plateau / top section of the EP.** Place a sheet cross wise underneath the Elevation Pillow. If post intubation the patient needs to be moved down the O.R. table (into lithotomy position for example), by lifting with the sheet, the Elevation Pillow & patient can be moved as a unit and the patient remains in the same head elevated laryngoscopy position (H.E.L.P.).
- 2) Put the O.R. table in ~5-10 degrees back down position  
(The patient can then transfer from stretcher to O.R. table on a level plain.)
- 3) The patient's shoulders should be resting on the upper curve of the **Elevation Pillow**. The patient's head should rest on the standard intubating pillow at the upper edge of the Elevation Pillow.
- 4) After the patient is on the O.R. table and resting on the **Elevation Pillow & standard intubating pillow**, adjust the O.R. table ~5-10 degrees back-up as needed.
- 5) Build-up the arm boards with foam and / or folded blankets or place the **Elevated (vinyl covered) Arm Board pads (sold separately)...item # 10-910-07**
- 6) **Place a 'regular' hospital pillow under the patient's knees.**

**CAUTION:** If a steep head down position is planned for the surgery; it is recommended that the Elevation Pillow **be removed** post intubation.

**CLEANING:** Use Virex or Clorox Bleach disinfectant products such as Fantastik, Formula 409

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(Manufacturer)  
CR Enterprises, LLC  
4701 Augusta Drive  
Frisco, Texas 75034

Scanlon Group BV in the Netherlands  
Tel: + 353 91 443609

Distributed by: Mercury Medical; Clearwater, Florida USA

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(Manufacturer)

CR Enterprises, LLC  
4701 Augusta Drive  
Frisco, Texas 75034

Scanlon Group BV in the Netherlands  
Tel: + 353 91 443609

Distributed by: Mercury Medical; Clearwater, Florida USA

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  
(Manufacturer)  
CR Enterprises, LLC  
4701 Augusta Drive  
Frisco, Texas 75034

EC REP

Scanlon Group BV in the Netherlands  
Tel: + 353 91 443609

Distributed by: Mercury Medical, Clearwater, Florida USA

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