(For use with **FLWSAFE** **FLWSAFE II** **FLWSAFE II EZ**)

INTENDED USE:

The Anti-Asphyxia Valve Elbow is an accessory to the Flow-Safe (CPAP) family of products for use with CPAP and Bi-level systems.

CAUTIONS:

- This device should not be used on uncooperative or unresponsive patients.
- This device may not be suitable for use on patients unable to remove the mask or are taking medication which may cause vomiting.
- This device is designed to be used in conjunction with Mercury Medical Flow-Safe masks.

WARNINGS:

- This elbow is non vented; if used in a system requiring an exhalation/leak port, a separate exhalation/leak device must be included in the circuit.
- Incorrect operation of this device can be hazardous.
- Not intended for use with systems exceeding 60 cm H₂O.
- Not intended for use with systems that do not maintain positive pressure.
- Do not leave patients unattended when device is in use.
- Do not block the Anti-Asphyxia Valve vent hole.
- Do not allow smoking or use unit near sparking equipment, open flame, oil or other flammable chemicals.
- This accessory must be used in the orientation labeled on the device.
- Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

Anti-Asphyxia
Valve Elbow

DIRECTIONS FOR USE:

- Remove Mercury Medical CPAP device from mask.
- Insert Anti-Asphyxia Valve Elbow 22 mm male port into mask as indicated by graphic above.
- Gently press the Anti-Asphyxia Valve Elbow into the mask until fully seated. Do not use excessive force.
- Connect Circuit to 22 mm female port.

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