Baby Tape PLUS™

Hydrocolloid Endotracheal Tube, NG/OG, and Cannula Holder For Infants

Contents: 2 UNITS

REF 25390



LOT

② — Disposable —
Single Patient Use Only
DO NOT REUSE

Not Made With
Natural Rubber Latex

CAUTION: Federal law
restricts this device to sale by
or on order of a physician.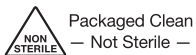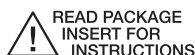


Manufactured for:
B&B Medical Technologies
1954 Kellogg Ave. Ste 100
Carlsbad, CA 92008 U.S.A
Phone: +1.760.929.9972
Fax: +1.760.929.9953
www.bandb-medical.com

EC REP **EU Representative:**
Scanlan Group B.V.
Postbus 75664
1118 ZS Schiphol Triport
The Netherlands
Phone: 31(0) 20-653-0553
Fax: 31(0) 20-653-3053

Made in USA
Assembled in Mexico
Form S539U Rev 1.0

Package Includes:

2 each Baby Tape, 2 each Strips of NG, OG and Cannula Security Tape

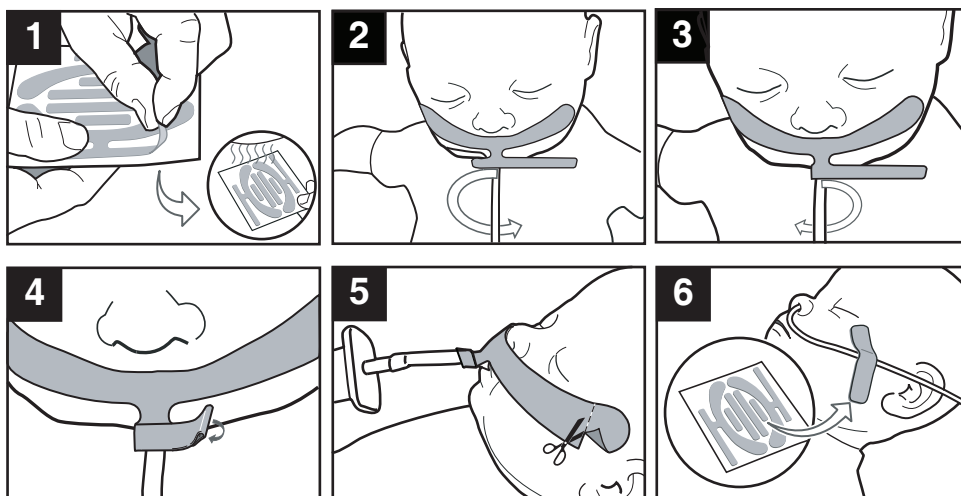Baby Tape PLUS™

Infant Endotracheal Tube Holder and NG & OG Tube Security

**READ AND UNDERSTAND COMPLETE
INSTRUCTIONS PRIOR TO USE**

1. Clean Infant's skin and dry. Skin prep pads may be applied to clean and prep the application area and the endotracheal tube. Fan dry for 20 seconds.
2. Peel the Baby Tape Plus from the card (See Figure 1). Align the center of the small tabs directly above the endotracheal tube (See Figure 2).
3. Secure the Baby Tape Plus by starting under the nose, gently pressing the large wings to the infant's upper lip and cheeks in an outward direction. Apply gentle pressure with warm hands to the upper lip and cheeks to initiate adhesion. Hold in place for 30-45 seconds.
4. Wrap the short tab counter clockwise around the endotracheal tube (See Figure 2).
5. Wrap the long tab clockwise around the endotracheal tube, overlapping the first short tab (see Figure 3). Make a small release tab at the end of the long tab to ease removal by folding over the edge (See Figure 4).
6. The Baby Tape Plus can be trimmed to fit the baby's cheeks (See Figure 5).
7. Periodically inspect the Baby Tape Plus. Replace every 48 hours or more frequent if soiled or the adhesion is compromised.
8. The center strips of Baby Tape Plus may be used to secure NG, OG and nasal cannula's to the infant (See Figure 6). Trim the strip to fit the tube. Place the tape on clean, dry skin and adhere the tube.

Spanish, German, French on reverse side

Caution:

Do not use water based skin prep to clean the skin prior to application. Moisture, oils or lotions may adversely affect the adhesion of the Baby Tape Plus. Discontinue immediately if skin irritation occurs. Any serious incident that has occurred in relation to this device should be reported to the manufacturer and competent authority of the Member State in which the user and/or patient is established

CAUTION: Federal (USA) law restricts this device to sale on or by a physician or other licensed practitioner. Single Patient use only. Disposable. Not Sterile. DO NOT REUSE.

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