

CPR-2 BAG

Quick Steps:

1. Pull Red Tab to Activate Light.
Light Flashes Every 6 Seconds
2. Pop-Off is Set at 40 cmH₂O
+/- 5 cmH₂O. For Increased
Pressure, Engage Lock-Clip
Upward Over Button
3. Adjust to Desired PEEP
To Increase PEEP, Turn Clockwise
To Decrease PEEP, Turn Counter-
clockwise
4. Monitor Manometer for
Appropriate Airway Pressure
& PEEP Pressure

