Deluxe Full Face Mask

Single-Patient-Use

(en)

Fulde Deluxe-Ansigtsmaske

Kun til brug med en enkelt patient

(da)

Volledige Gezicht Deluxe Masker

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(nl)

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Usage unique

(fr)

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(de)

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(it)

INDICATIONS FOR USE:

The intended use of the Deluxe Full Face Mask is to direct gas to the upper airway.

CAUTIONS:

1. During use, the patient's condition must be continuously monitored.
2. The Mask should only be used with CPAP, non-invasive ventilation devices or as recommended by attending clinician.

DIRECTIONS FOR USE:

1. Connect to device with appropriate fitting.
2. Place and position Mask on patient.
3. Connect head straps with quick connect clips to Mask.
4. Adjust hook and loop headgear straps and foam forehead pad position to properly fit patient.
5. Inspect all connections & fittings to ensure secure connections.
6. Dispose of according to facility protocol.

WARNINGS:

- In the USA, federal law restricts this Mask to sale by, or on the order of a physician. (Rx ONLY)
- This Mask does not have an integral leak port; therefore, if required, a separate exhalation device must be part of the circuit. Utilize the equipment manufacturer's recommended exhalation port in the designated circuit.
- If the patient experiences skin irritation, consult physician.
- This Mask is not suitable for providing mechanical life support ventilation.
- Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.
- Do not clean, soak, rinse or sterilize. To do so may result in harmful residuals or have adverse side effects on this product.
- Reuse of this device may pose a risk of cross-contamination and the device may not perform as intended.
- This mask is nonconductive and should not be used in the presence of flammable anesthetics.

SPECIFICATIONS:

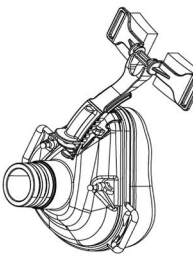
MR Conditional: Static magnetic field of 3.0 Tesla or less
Spatial Gradient of 720 gauss/cm or less

The MERCURY MEDICAL® Deluxe Mask was designed to meet ISO 5356-1 requirements.

Port Size: 22 mm I.D.

90° Swivel Port and Anti-asphyxia Valve

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