## Double Swivel Elbow Connector

(en)

## Dobbelt, drejelig knæforbinder

(da)

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(it)

### INDICATIONS FOR USE:

The double swivel elbow connector is indicated for temporary connection to mechanical ventilator or anesthesia circuits to the artificial airway. The connector will help to eliminate torque and undesirable motion.

### CONTRAINDICATIONS:

Swivel elbow connectors should not be used for administration of continuous flow oxygen or other gases.

### WARNINGS:

- Continuous flow gas administration through either the side arm or the bronchoscopic cap (if equipped) may result in pulmonary barotrauma or iatrogenic pneumothorax.
- Circuits should be well supported to help reduce lateral stress on the swivel elbow connector. Excess stress may compromise the integrity of the connector.
- Inadvertent dislodgement of the bronchoscope cap (if equipped) or connector will result in loss of tidal volume.
- Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

### PRECAUTIONS:

- Follow standard infection control procedures as specified by the Centers for Disease Control and Prevention (USA) or local equivalent.
- Disposable for single use only.

### INSTRUCTIONS FOR USE:

1. Remove the connector from its package and occlude the opening with the attached plug.
2. Attach to the patient's circuit and artificial airway.
3. Support the circuit to reduce lateral stress to the connector.
4. If supplied with swivel elbow, remove the occlusive plug from the connector.
5. If equipped, open the cap by removing the plug and pass the bronchoscope through the fenestra into the airway.
6. Following the procedures, replace the plug on the cap.

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