MercuryMedical
11300 - 49th Street North
Clearwater, Florida 33762-4807 USA
www.mercury-med.com

**Flow-Safe II® &
Flow-Safe II EZ®**
Apnea Alarm Accessory
Single-Patient-Use

(en)

CONTENTS:
In-Line Tee
Luer Lock Sampling Tubing

INSTRUCTIONS:
Insert In-Line Tee between mask and Flow-Safe II & Flow-Safe II EZ device. Securely connect Luer Lock Sampling Tubing to In-Line Tee and the anesthesia system's gas analyzer.

CAUTIONS:
1. Single use only. Do not sterilize or re-use to reduce the risk of possible transmission of infection.
2. During use, the patient's condition must be continuously monitored.
3. Federal (USA) law restricts this device to sale by or on the order of a physician.

WARNINGS:
- For apnea alarm triggering only. If used for capnometry or capnography, numeric value or wave form may be attenuated. Do not use as a diagnostic tool. For accurate capnometry or capnography, a nasal sampling cannula is recommended.
- Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

2797
EMERGO EUROPE
Prinsessegracht 20
2514 AP The Hague
The Netherlands

 Do Not Reuse
 Non Sterile
 DINP
 Not Made with Natural Rubber Latex
 Rx
By Prescription Only

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