Mercury Medical **FLOWSAFE II** RESPONDER CPAP SYSTEM
Low FiO₂ Elbow (en)

11300 - 49th Street North
Clearwater, Florida 33762-4807 USA
www.mercury-med.com

INTENDED USE:
The Low FiO₂ Elbow is an accessory to the Flow-Safe II (CPAP) family of products to reduce the FiO₂ to approximately 35%.

CAUTIONS:

1. This device should not be used on uncooperative or unresponsive patients.
2. This device may not be suitable for use on patients unable to remove the mask or are taking medication which may cause vomiting.
3. This device is designed to be used in conjunction with Mercury Medical Flow-Safe masks.
4. This device is designed to only be used in conjunction with Mercury Medical Flow-Safe II CPAP systems.
5. The Low FiO₂ Elbow reduces oxygen concentration to approximately 35% during laboratory testing. Patient characteristics and gas supply flow rate can affect the oxygen concentration delivered.
6. The Low FiO₂ Elbow is not designed to be used with a nebulizer. Oxygen concentration may be greater than 35% and medication may be lost to the environment if a nebulizer is introduced to the system.
7. The Low FiO₂ Elbow is designed to be used with the Flow-Safe II with oxygen supply flow rates greater than 8 LPM.

WARNINGS:

1. Incorrect operation of this device can be hazardous.
2. Not intended for use with systems that do not maintain positive pressure.
3. Do not leave patients unattended when device is in use.
4. Do not block the Anti-Asphyxia Valve vent hole.
5. Do not allow smoking or use unit near sparking equipment, open flame, oil or other flammable chemicals.
6. This accessory must be used in the orientation labeled on the device.
7. Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

Low FiO₂ Elbow

DIRECTIONS FOR USE:

1. Remove Mercury Medical CPAP device from mask.
2. Insert Low FiO₂ Elbow 22 mm male port into mask as indicated by graphic above.
3. Gently press the Low FiO₂ Elbow into the mask until fully seated. Do not use excessive force.
4. Connect Flow-Safe II to 22 mm female port.

CE 2797 EMERGO EUROPE Prinsessegracht 20 2514 AP The Hague The Netherlands

Do Not Reuse Non Sterile Not Made with Natural Rubber Latex By Prescription Only

EC REP

Mercury Medical **FLOWSAFE II** RESPONDER CPAP SYSTEM
Low FiO₂ Elbow (da)

11300 - 49th Street North
Clearwater, Florida 33762-4807 USA
www.mercury-med.com

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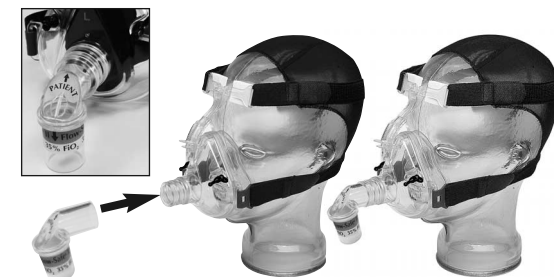
EC REP

Mercury Medical **FLOWSAFE II** RESPONDER CPAP SYSTEM
Lage FiO₂ Elleboog (nl)

11300 - 49th Street North
Clearwater, Florida 33762-4807 USA
www.mercury-med.com

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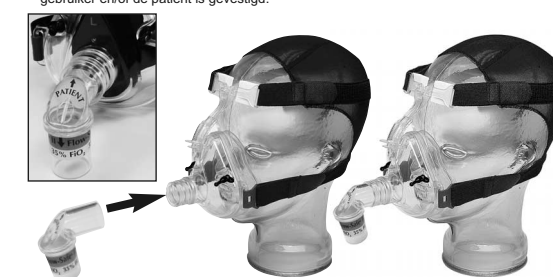
EC REP

Mercury Medical **FLOWSAFE II** RESPONDER CPAP SYSTEM
Low FiO₂ Elbow (fi)

11300 - 49th Street North
Clearwater, Florida 33762-4807 USA
www.mercury-med.com

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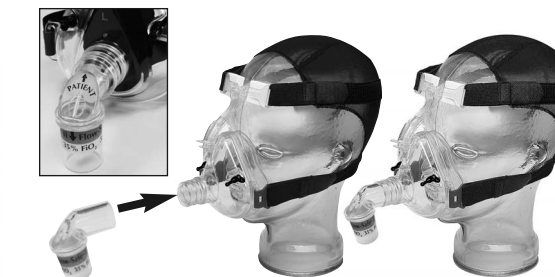
EC REP

Mercury Medical **FLOWSAFE II** RESPONDER CPAP SYSTEM
Low FiO₂ Elbow (coude bas FiO₂) (fr)

11300 - 49th Street North
Clearwater, Florida 33762-4807 USA
www.mercury-med.com

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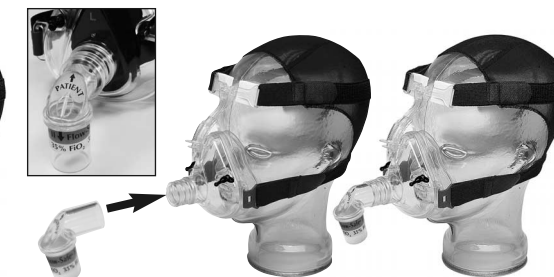
EC REP

Mercury Medical **FLOWSAFE II** RESPONDER CPAP SYSTEM
Low FiO₂ Elbow (de)

11300 - 49th Street North
Clearwater, Florida 33762-4807 USA
www.mercury-med.com

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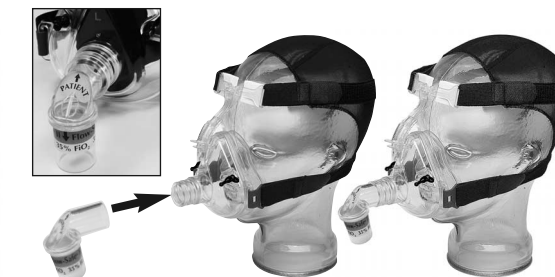
EC REP

Mercury Medical **FLOWSAFE II** RESPONDER CPAP SYSTEM
Low FiO₂ Elbow (el)

11300 - 49th Street North
Clearwater, Florida 33762-4807 USA
www.mercury-med.com

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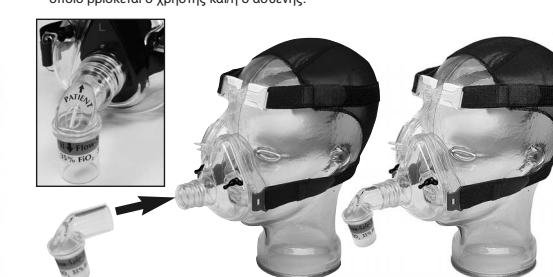
EC REP

Mercury Medical **FLOWSAFE II** RESPONDER CPAP SYSTEM
Low FiO₂ Elbow (it)

11300 - 49th Street North
Clearwater, Florida 33762-4807 USA
www.mercury-med.com

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