PEEP Valve
Single-Patient-Use

Disposable Positive End Expiratory Pressure (PEEP) Valve

INDICATIONS FOR USE:

The Mercury Medical PEEP Valve is a single patient use device that is attached to manual resuscitators and portable ventilators and is used to provide positive-end expiratory pressure. The device is adjustable from 0-20 cm H₂O pressure.

DIRECTIONS FOR USE:

1. Remove product from package.
2. Attach the Mercury Medical PEEP Valve to PEEP valve port. Connect test lung to patient port.
3. Adjust PEEP pressure (verify with manometer).
4. Dispose of according to facility protocol.

WARNINGS:

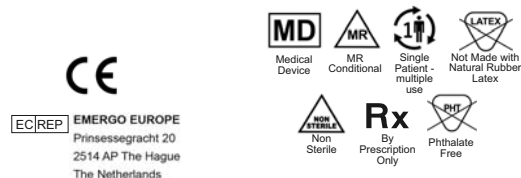
1. The Mercury Medical PEEP Valve should be used only by experienced persons who are aware that PEEP may have an adverse effect on the patient's circulation.
2. When using a Mercury Medical PEEP Valve, the patient's condition should be monitored and PEEP verified with a manometer.
3. Ensure the PEEP Flow direction is constant with the expiratory flow.
4. Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

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Specifications:

Range: 0-20 cm H₂O
Port Size: 30 mm Female Fitting (per ISO 5356-1)
Weight: 34 grams nominal
MR Conditional: Static magnetic field of 3.0 Tesla or less
Spatial Gradient of 3000 gauss/cm or less

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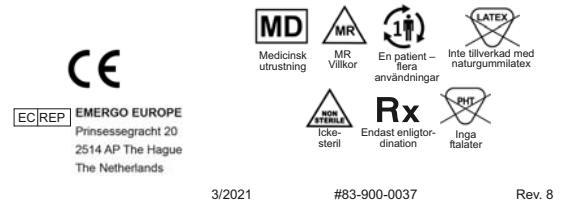
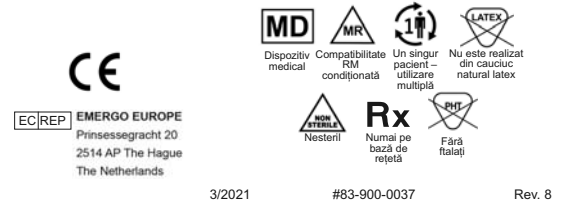
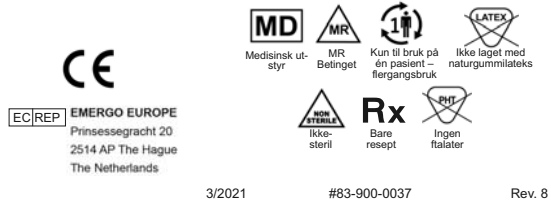
PEEP Valve with Filter

Single-Patient-Use

Disposable Positive End Expiratory Pressure (PEEP) Valve with Integral Filter

INDICATIONS FOR USE:

The Mercury Medical PEEP Valve with Filter is a single patient use device that is attached to manual resuscitators and portable ventilators and is used to provide positive-end expiratory pressure. The filter is intended to remove microbiological and particulate matter from exhaled gases. The device is adjustable from 0-20 cm H₂O PEEP.

DIRECTIONS FOR USE:

1. Remove product from package.
2. Attach the Mercury Medical PEEP Valve with Filter to PEEP valve port. Connect test lung to patient port.
3. Adjust PEEP (verify with manometer).
4. Dispose of according to facility protocol.

WARNINGS:

1. The Mercury Medical PEEP Valve with Filter should be used only by experienced persons who are aware that PEEP may have an adverse effect on the patient's circulation.
2. When using a Mercury Medical PEEP Valve with Filter, the patient's condition should be monitored and PEEP pressure verified with a manometer.
3. Ensure the PEEP flow direction is consistent with the expiratory flow.
4. Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

CAUTIONS:

1. Single use only. Do not sterilize or re-use to reduce the risk of possible transmission of infection. Discard after use.
2. Federal (USA) law restricts this device to sale by or on the order of a physician.
3. Reuse of this device may pose a risk of cross-contamination and the device may not perform as intended.

Specifications:

PEEP Valve:	
Range:	0-20 cm H ₂ O
Port Size:	30 mm Female Fitting (per ISO 5356-1)
MR Conditional:	Static magnetic field of 3.0 Tesla or less
	Spatial Gradient of 3000 gauss/cm or less

Filter:

Bacterial Filter Efficiency (BFE)/Viral Filter Efficiency (VFE)

Mean Efficiency:	>99.9% effective on particles with an approximate size of 3.2 microns
Resistance to Flow:	1.02cm H ₂ O at 8.0 LPM Flow

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(it)

INDICATIONS FOR USE:

The Mercury Medical PEEP Adapter is designed to make PEEP (Positive End Expiratory Pressure) available for use on resuscitators. The adapter attaches first to the patient valve of the resuscitator and then accepts a PEEP valve having either a 22 mm male or 30 mm female connector.

DIRECTIONS FOR USE:

1. If the resuscitator has a mask attached, remove the mask.
2. Holding the patient valve in one hand, gently press and twist the adapter onto the patient valve with the other hand until the outer rubber ring of the adapter is fully seated onto the patient valve.
3. Rotate the swivel connector to assure that the center rubber bushing of the adapter is properly seated.
4. Attach PEEP valve using a twisting and pressing motion.
5. Dispose of according to facility protocol.

CAUTIONS:

- Follow PEEP valve manufacturer's instructions for PEEP application and contraindications when resuscitating a patient or administering CPAP therapy.
- Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.
- Reuse of this device may pose a risk of cross-contamination and the device may not perform as intended.

SPECIFICATIONS:

Port Size: 22 mm Male and 30 mm Female (per ISO-5356-1)
MR Conditional: Static magnetic field of 3.0 Tesla or less
Spatial Gradient of 3000 gauss/cm or less

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